



SafeTouch® II

SAFETY FISTULA NEEDLE



SafeTouch II之特色及優點

- 內建安全機制：永久上鎖之特點可降低意外針扎之擔憂
- 針頭可回收於安全的防護裝置
- 具止滑效果之蝴蝶翼，握住蝴蝶翼時更為穩固，插入導管時更為容易




**NIPRO CONNECT®
PROGRAM**


**Our commitment to strengthen
every partnership.**

All Nipro renal products are backed by the Nipro Connect Educational Program – offering clinicians product evaluations, in-services, continuing education units (CEU's) and instructional materials.


拔除SafeTouch II之方法

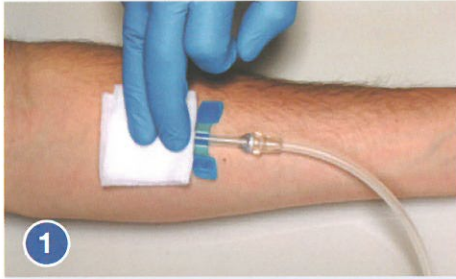


1
撕開蝴蝶翼及穿刺部位固定用的膠帶，於穿刺部位放上已滅菌之紗布。

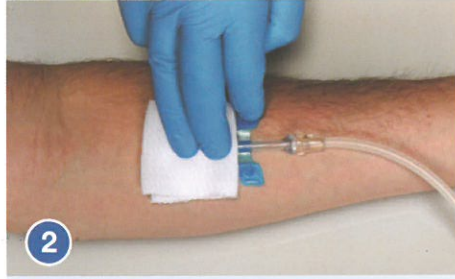

2
中指及無名指輕壓穿刺部位，並用食指固定蝴蝶翼。

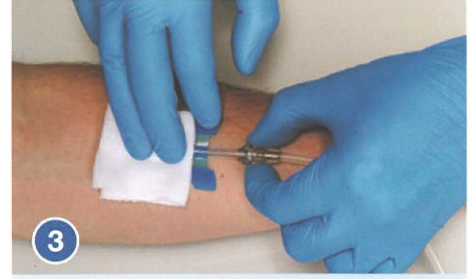

3
另一隻手之拇指及食指握住並加壓制動裝置，往後拉直到聽到喀嚓聲，表示針已安全鎖住。

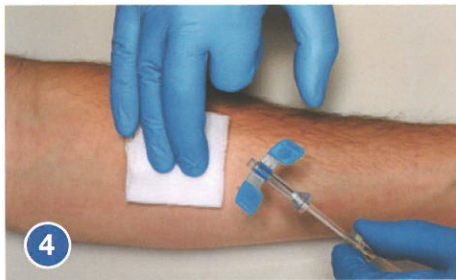

4
安全裝置已啟動完成，將針丟棄至適當的回收容器。

ORDERING INFORMATION

Item #	Description	Packaging
FS+142530BC	14GX1 Fixed Hub 12" Tubing	50/Box 10 Boxes/Case (500)
FS+152530BC	15GX1 Fixed Hub 12" Tubing	
FS+153230BC	15GX1.1/4 Fixed Hub 12" Tubing	
FS+162530BC	16GX1 Fixed Hub 12" Tubing	
FS+163230BC	16GX1.1/4 Fixed Hub 12" Tubing	
FS+172530BC	17GX1 Fixed Hub 12" Tubing	
FS+173230BC	17GX1.1/4 Fixed Hub 12" Tubing	
FS+173230BC-G	17GX1.1/4 Fixed Hub 12" Tubing - Gamma	
FS+142530BCR	14GX1 Rotating Hub 12" Tubing	
FS+152530BCR	15GX1 Rotating Hub 12" Tubing	
FS+153230BCR	15GX1.1/4 Rotating Hub 12" Tubing	
FS+162530BCR	16GX1 Rotating Hub 12" Tubing	

Item #	Description	Packaging
FS+163230BCR	16GX1.1/4 Rotating Hub 12" Tubing	50/Box 10 Boxes/Case (500)
FS+172530BCR	17GX1 Rotating Hub 12" Tubing	
FS+173230BCR	17GX1.1/4 Rotating Hub 12" Tubing	
FS+152530CR	15GX1 Rotating Hub w/o Backeye 12" Tubing	
FS+162530CR	16GX1 Rotating Hub w/o Backeye 12" Tubing	
FS+172530CR	17GX1 Rotating Hub w/o Backeye 12" Tubing	
FS+142530TP	14GX1 Twin Pack 12" Tubing	2/Package Red – Arterial Blue – Venous
FS+152530TP	15GX1 Twin Pack 12" Tubing	
FS+162530TP	16GX1 Twin Pack 12" Tubing	
FS+172530TP	17GX1 Twin Pack 12" Tubing	25 Pairs/Box 10 Boxes/Case (250 Pairs)
FS+152530TPR	15GX1 Twin Pack Rotating Hub 12" Tubing	
FS+162530TPR	16GX1 Twin Pack Rotating Hub 12" Tubing	
FS+172530TPR	17GX1 Twin Pack Rotating Hub 12" Tubing	

CAUTION: Federal law restricts these devices to sale by or on the order of a physician.



華江醫療儀器股份有限公司
Hua Chiang medical co., ltd

【台北地區】
地址：台北市羅斯福路三段245號4F之2
電話：(02)23620396~7、傳真：02-23620398
Email：taipei@huachiang.com

【台南地區】
地址：702台南市南區新榮路23-1
電話：06-2647878-0、傳真：06-2636966
Email：tainan@huachiang.com