


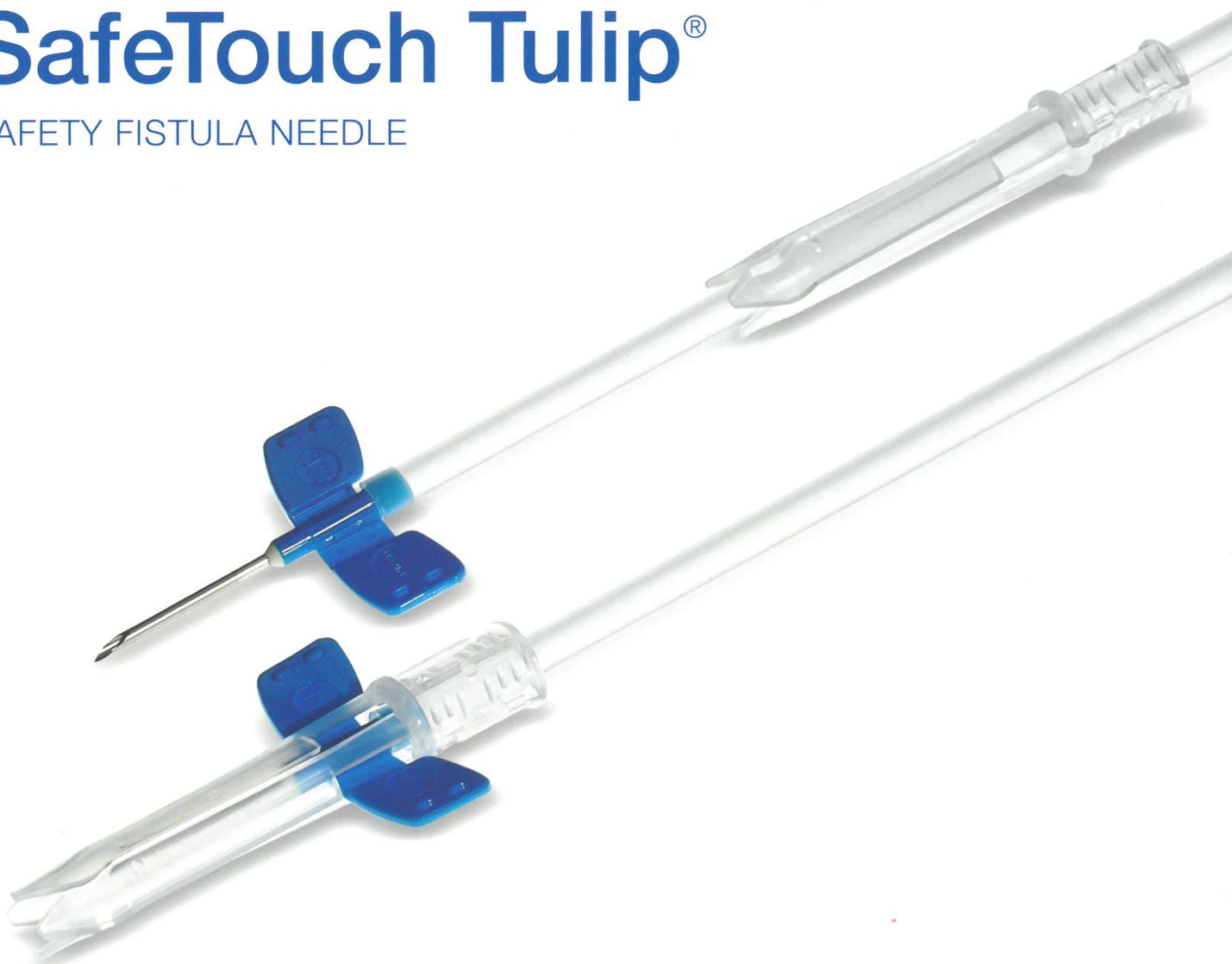

SafeTouch Tulip®

SAFETY FISTULA NEEDLE



SafeTouch Tulip 之特色及優點

- 病患插管及上針時操作方便
- 簡易的安全機制，單或雙手皆可操作
- 喀噠聲表示安全裝置牢固鎖上




**NIPRO CONNECT®
PROGRAM**

**Our commitment to strengthen
every partnership.**

All Nipro renal products are backed by the Nipro Connect Educational Program – offering clinicians product evaluations, in-services, continuing education units (CEU's) and instructional materials.

www.nipro.com

拔除安全前推式內瘻管翼狀針之方法

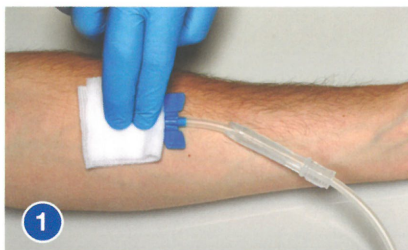


1

撕除雙翼及注射部位周圍的膠帶，將已滅菌的紗布覆蓋於穿刺部位，將針的安全機制朝雙翼方向調整。




2

壓住紗布，將兩指置於安全機制末端較硬的部分。往後拉翼狀針，直到兩個花瓣部分覆蓋住雙翼。注意：兩指的部分請握於tubing下方。

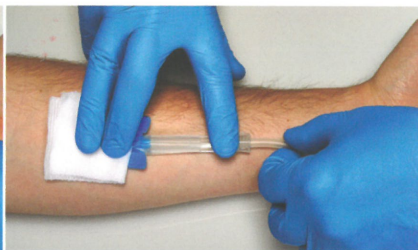

3

牢固的壓住安全機制後，再次將tubing往後拉，直到雙翼完全被扣住，並聽到喀噠聲確認安全機制已牢固鎖上。

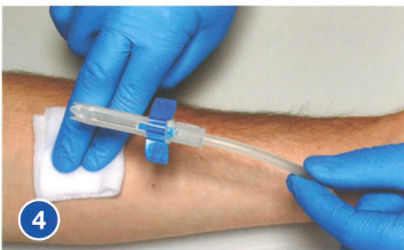

4

將本產品丟棄至可回收尖銳物品之容器。

ORDERING INFORMATION

Wing color	Description	Tube Length	Packaging
 黃	15GX1	30公分	單支包裝， 50支／盒， 10盒／箱。
	15GX1-1/4		
	16GX1		
	16GX1-1/4		
	17GX1		
	17GX1-1/4		



華江醫療儀器股份有限公司
Hua Chiang medical co., ltd

【台北地區】
台北辦事處：台北市羅斯福路三段245號4F-2
電話：(02)2362-0396-7
傳真：(02)2362-0398
Email：taipei@huachiang.com

www.hcmed.com.tw

【台南地區】
台南總公司：702台南市南區新孝路197號
電話：(06)264-7878-0
傳真：(06)263-6966
Email：tainan@huachiang.com